



Outstanding in Infection Control with SFED

SteriMicro™ - QAC

3rd Generation QAC For Air and Surface Disinfection.

Aldehyde free, alcohol free, excellent broad-spectrum disinfectant, including antibiotic resistant micro-organism strains. It has a powerful cleaning effect, very good material compatibility and is a 3rd generation QAC.



PRODUCT PROPERTIES

- Broad Spectrum Disinfectant.
- Bactericidal, Fungicidal, Virucidal, including Antibiotic Resistant Strains - MRSA & VRE.
- Excellent Cleaning Power
- Excellent Material Compatibility.
- Corrosion Free
- Biodegradable – Eco-Friendly.
- Colorless
- Non-toxic

COMPOSITION

N-Alkyl dimethyl benzyl ammonium chloride
2.37%
N-Alkyl dimethyl ethyl benzyl ammonium chloride
2.37%



CE



QMS
INTERNATIONAL



STERIMICRO™

CRITICAL AREAS:

All ICU's, OT's, AKD, CCU, Oncology, Cath Lab etc.

NON CRITICAL :

OPD's, Laboratories, Dental Clinic's, Wards, Corridors, Casualties etc.

Outstanding in Infection Control with SFED

SteriMicro™ – QAC

3rd Generation QAC For Air and Surface Disinfection.



AREAS OF APPLICATION

- SteriMicro™-QAC is suitable for the disinfection of Air and washable surfaces using the wet-wipe-procedure.
- For medical equipment which come under the Medical Device Directive (acc. to MDD)
- In hospitals and residential homes.
- In industrial kitchens and food-processing areas.

MATERIAL COMPATIBILITY

Metals : Stainless steel (V2A), aluminium, copper, brass. **Plastics :** Polyamide, polyethylene, polypropylene, polystyrene, polyurethane, PVC, acryl-butadiene-styrole, silicone, rubber, latex, Makrolon(R), acrylic glass, Teflon(R), Vivak(R) clear 099.

MOPPING (WET WIPE) IS RECOMMENDED OVER FOGGING

CDC - USA - Guidelines: 2003, 2008. "Do not perform disinfectant fogging for routine purposes in patient-care areas." Category IB, Category II.

	APPLICATION	DILUTION RATIO
Air Disinfection (Fogging)	Closed area in absence of personnel 1L of diluted solution for 1000 cubic feet area over 10 minutes.	15ml SteriMicro™-QAC in 1 Liter water
Critical Surface	High risk areas such as ICU, ICCU, SICU, Gynecology, Neonatal, Burns Oncology, Dialysis, Operation Theatres etc.	15ml SteriMicro™-QAC in 1 Liter water
Non-Critical Surface	Intermediate / Low risk areas (Floor, Walls), OPD's Clinics Mobile units, Dental clinics, Wards etc.	7.50 ml SteriMicro™-QAC in 1 Liter water
Fabric / Linen Disinfection	For manual was Soak & rinse with water in washing machine and add to fabric conditioner.	15ml SteriMicro™-QAC in 1 Liter water for 4 kg load.
Devices Disinfection	Clean devices & soak in SteriMicro™- QAC for minimum 10 minutes.	15ml SteriMicro™-QAC in 1 Liter water
SteriMicro™- QAC is compatible with hard water, up to a TDS of 800 ppm.		
Self-Life - 3 Years in packed condition and 64 days in diluted condition.		

PACKING : 1 Ltr.

SFED India Pvt. Ltd.

ISO 9001: 2015, GMP, CE & ISO 13485: 2016 Compliance Company
SF-221, 2nd Floor, IT Complex, JMD Megapolis, Sohna Road, Gurgaon, Haryana-122018
Contact: - 9026326293; Mail - info@sfedindia.in
www.sfedindia.in